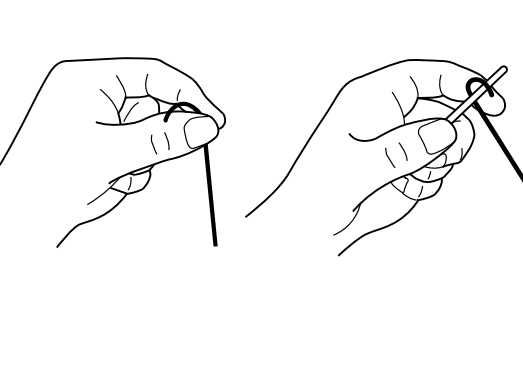
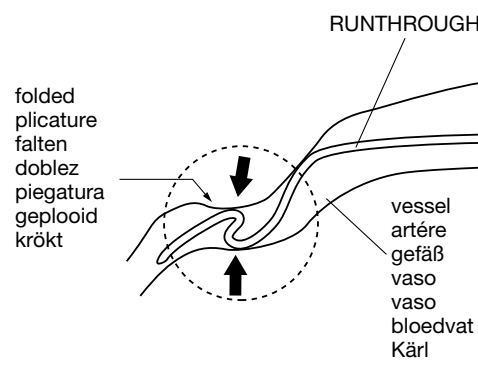
RUNTHROUGH NS

PTCA Guide Wire / Guide PTCA / PTCA draht / Guía ACTP Guída per PTCA / PTCA Voerdraad / PTCA Guide Wire

Fig. 1 / Fig. / Abb. / Fig. /
Fig. 1 / Fig. / FigurFig. 2 / Fig. / Abb. / Fig. /
Fig. 2 / Fig. / Figur

Rx ONLY

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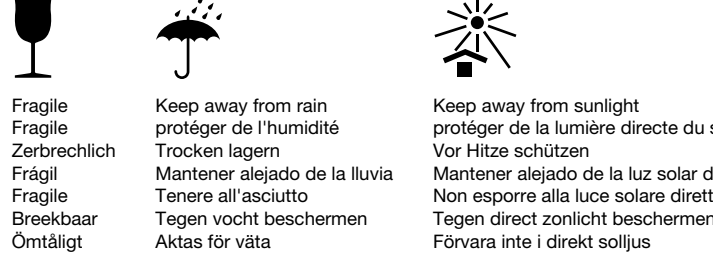
ENGLISH

INSTRUCTIONS FOR USE

Read the following warnings, precautions, and directions for use carefully.
CAUTION: Federal law (U.S.A.) restricts this device to sale by or on the order of a physician.

INDICATIONS

The RUNTHROUGH NS are used to facilitate the placement of balloon dilatation catheters for percutaneous transluminal coronary angioplasty (PTCA) and/or percutaneous transluminal angioplasty (PTA).

DESCRIPTION

The RUNTHROUGH NS is a guide wire in use in PTCA with good conformability and trackability in the vessels. The distal part of the wire has a spring coil with hydrophilic polymer coating which generates high lubricity when wet. The distal portion of the spring coil has a radiopaque spring coil which provides good visibility and shapeability.

CONTRAINDICATIONS

- Patients who had previous coronary artery spasm.
- Pregnant patients or patients who may be pregnant.
- Patients who, because of their condition, are contraindicated for surgical operation.
- Lesions that follow the bypass constructed less than one month ago or the false aneurysm.
- Patients who are contraindicated for anti-platelet or anti-coagulation treatment.
- Patient who has a critical allergy to contrast media.
- Caution: Read the instructions accompanying the other interventional devices to be used with the RUNTHROUGH NS.

COMPLICATIONS

- Possible complications associated with the use of the RUNTHROUGH NS include, but are not limited to, the following:
 - Vessel perforation
 - Initial tear
 - Vessel damage
 - Vessel spasm
 - Haematoma at the puncture site
 - Infections
 - Cardiac perforation
 - Kidney perforation
 - Pain
 - Distal embolus
 - Temporary modification of blood flow

DIRECTIONS FOR USE

- Preparation of the RUNTHROUGH NS
 - Remove the RUNTHROUGH NS and the holder from the package.
 - Before removing the RUNTHROUGH NS from the package, inject heparinized physiological saline solution through the hub of the holder with a syringe.
 - Detach the proximal end of the RUNTHROUGH NS from the holder tube clip, and slowly remove the RUNTHROUGH NS from the holder tube. Gently remove the RUNTHROUGH NS from the wire clip.
 - Soak the RUNTHROUGH NS sufficiently in heparinized physiological saline solution.
 - WARNING: The proximal end of the RUNTHROUGH NS exposes from the holder. Make sure of the direction of the guide wire when inserting the guide wire into the dilatation catheter.
- Distal part of the RUNTHROUGH NS is not lubricious unless it is wet. Damage to the vessel or the catheter and/or separation of the guide wire's tip may result if it is handled with its surface dry or not sufficiently wet.
 - Prior to use, wet the surface of the RUNTHROUGH NS and the distal part of the RUNTHROUGH NS is not flexible unless it is handled with its surface sufficiently wet.
 - The distal part of the RUNTHROUGH NS may be curved due to the hydrophilic polymer when taken out from the holder tube unless it is wet. It can be straightened to the original figure when its surface is wetted sufficiently.
 - Do not resist the RUNTHROUGH NS into the holder tube. It may result in damage to or the release of hydrophilic polymer fragments.
- Advance the dilatation catheter to the desired lesion over the RUNTHROUGH NS.
 - CAUTIONS:
 - Damage to or separation of the guide wire may result if it is handled with its surface dry, heated, held with fingernails, forceps, or its other dry, hard, bent sharply, or bent repeatedly at the same point.
 - The shapeability of the RUNTHROUGH NS may be insufficient unless it is wetted sufficiently.
 - If not intended to insert the RUNTHROUGH NS into a dilatation catheter immediately, keep the RUNTHROUGH NS in the holder with heparinized physiological saline solution till the insertion.
- Insertion into a balloon dilatation catheter
 - CAUTION:
 - Damage to or separation of the guide wire may result if it is handled with its surface dry, heated, held with fingernails, forceps, or its other dry, hard, bent sharply, or bent repeatedly at the same point.
 - The sheapability of the RUNTHROUGH NS may be insufficient unless it is wetted sufficiently.
 - If not intended to insert the RUNTHROUGH NS into a dilatation catheter immediately, keep the RUNTHROUGH NS in the holder with heparinized physiological saline solution till the insertion.

removed from the patient's vessel. Using a retrieving device while the RUNTHROUGH NS is in the vessel may cause the RUNTHROUGH NS to break.

- All devices used with the RUNTHROUGH NS should be sufficiently wetted prior to and during use with heparinized physiological saline solution. Improper priming of the system may result in damage to or the release of hydrophilic polymer fragments, and may prevent smooth movement of the wire.
- Do not use the RUNTHROUGH NS for intervention or for insertion into the vessel.
- Do not use the RUNTHROUGH NS as outer guide wire in case of stenting in the percutaneous technique.
- Do not bend the RUNTHROUGH NS repeatedly at the same point. It may result in deformation, breakage, or separation of the RUNTHROUGH NS.
- If catheter is used with a stent, do not operate this stent while the RUNTHROUGH NS is in the catheter. It may result in breakage or separation of the RUNTHROUGH NS.
- Manipulate the RUNTHROUGH NS carefully when crossing it through a deployed stent. Stent migration, stent deformation, or breakage or separation of the RUNTHROUGH NS may be caused if the RUNTHROUGH NS is caught by the deployed stent.
- The RUNTHROUGH NS should be used by, or under the order of a physician who is familiar with the indication and observation of guide wires under high resolution fluoroscopy.
- The RUNTHROUGH NS has been sterilized by ethylene oxide gas. For single use only. Do not resterilize or reuse. Do not use an unopened and undamaged unit package. Do not use the unit package or the RUNTHROUGH NS is damaged or soiled.
- The RUNTHROUGH NS should be used immediately after opening the packaging and be disposed of safely and properly after use, following local regulations for medical waste management.
- The entire procedure should be carried out aseptically.
- When using a drug or a device concurrently with the RUNTHROUGH NS, the operator should have a full understanding of the properties/characteristics of the drug or device so as to avoid damage to the RUNTHROUGH NS.

Use an appropriate balloon dilatation catheter corresponding to the outer diameter (O.D.) of the RUNTHROUGH NS. Due to the variation of center catheter tip inner diameters, abrasion of the hydrophilic coating may occur during manipulation. If any resistance is felt during introduction of the guide wire into the catheter, it is advisable to stop using such catheters.

- Inspect and prepare the dilatation catheter according to the instructions for use of the balloon dilatation catheter.
- If necessary, connect a Y connector to the guide wire port of the balloon dilatation catheter, and prime the balloon dilatation catheter with heparinized catheter with heparinized physiological saline solution if any blood residues still adhere to the guide wire.
 - WARNING: Prime the balloon dilatation catheter enough so that no air bubbles remain inside. Improper priming may prevent smooth movement of the guide wire.
- Make sure that the surface of the RUNTHROUGH NS is wet enough. Insert the RUNTHROUGH NS into the dilatation catheter and advance the guide wire slowly until guide wire and catheter tip are appropriately positioned. When a rapid exchange type balloon dilatation catheter is used, insert the RUNTHROUGH NS's proximal end into the distal tip of the catheter. Advance the guide wire through the guide wire lumen until guide wire and catheter tip are appropriately positioned.
 - CAUTION: Insertion of the RUNTHROUGH NS into the balloon dilatation catheter while bent may result in damage to the catheter by the proximal end of the guide wire.
- Tighten the haemostatic valve of the Y connector while verifying that the RUNTHROUGH NS's proximal end protrudes and moves smoothly.
 - CAUTION: Do not manipulate the RUNTHROUGH NS when strongly locked by the haemostatic valves as breakage of the RUNTHROUGH NS may result.

3. System operation

- Position the guiding catheter at the desired lesion using accepted protocol.
- Insert the dilatation catheter through the haemostatic valve of the Y connector attached to the guiding catheter placed to the desired lesion, and advance the dilatation catheter up to 1-2 cm from the distal end of the guiding catheter under high resolution fluoroscopy to prevent the tip from being trapped in the lesion.
- Contrast the RUNTHROUGH NS in multi-directions.
 - WARNING: When selecting the vessel in which the RUNTHROUGH NS is to be inserted or when advancing the guide wire through the stenosis, do not turn the proximal end of the guide wire three or more turns in succession in the same direction if the guide wire's tip is trapped. Separation of the guide wire may result.
 - If any resistance to the RUNTHROUGH NS or the dilatation catheter is felt during manipulation or if the shape, behavior or position of the guide wire's tip seems improper, e.g. in a case where the tip is trapped due to vessel spasm or some other cause or the tip is locked as shown in Fig. 2, stop manipulating the guide wire (and the catheter) and determine the cause of the resistance by fluoroscopy and take suitable remedial actions. Then remove the guide wire slowly, without turning, to the proximal end of the catheter. Do not use force or catheter manipulation in the said situations may result in damage to the vessel, damage to or separation of the guide wire's tip, and/or damage to the dilatation catheter.

4. When exchanging the balloon dilatation catheter

- CAUTION: Carefully refer the following directions and the instructions manual accompanying the balloon dilatation catheters.

180 cm guide wire

- Using with an "Over-The-Wire" dilatation catheter >
 - Prior to exchanging the dilatation catheter replace the 180 cm RUNTHROUGH NS with an exchange length guide wire following the directions for use labelled "wire exchange" in the RUNTHROUGH NS' user 4. Then refer to the instructions of the paragraph "Using with "Over-The-Wire" dilatation catheter" of the "300 cm".
- Do not exchange the dilatation catheter while flushing contrast media or heparinized physiological saline solution by a syringe or an injector and while the guide wire is kept in place. The RUNTHROUGH NS may be bent or broken.

Using with a "Rapid Exchange" dilatation catheter >

- Maintain the position of the RUNTHROUGH NS and carefully remove/insert the dilatation catheter while confirming the position of the guide wire's tip under high resolution fluoroscopy.

PRECAUTIONS FOR STORAGE

- Avoid exposure to water, direct sunlight, extreme temperatures and high humidity during storage. Store under controlled room temperature.

WARNING: When selecting the vessel in which the RUNTHROUGH NS is to be inserted or when advancing the guide wire through the stenosis, do not turn the proximal end of the guide wire three or more turns in succession in the same direction if the guide wire's tip is trapped. Separation of the guide wire may result.

WARNING: When selecting the vessel in which the RUNTHROUGH NS is to be inserted or when advancing the guide wire through the stenosis, do not turn the proximal end of the guide wire three or more turns in succession in the same direction if the guide wire's tip is trapped. Separation of the guide wire may result.

- When exchanging the RUNTHROUGH NS
 - Loosen the haemostatic valve of the Y connector, and remove the RUNTHROUGH NS slowly while confirming the position under high resolution fluoroscopy. Carefully exchange the guide wire so that no air enters the dilatation catheter.
 - Gently insert new RUNTHROUGH NS into the dilatation catheter in accordance with the above-mentioned Items 2-3 and 2-4. In case of using a Rapid Exchange type dilatation catheter, remove the dilatation catheter and the guide wire.
 - If resistance is felt during the RUNTHROUGH NS insertion, stop manipulation and exchange it for a new one. Manipulation against resistance may result in damage to the vessel, breakage of the RUNTHROUGH NS or the dilatation catheter.

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