RADIFOCUS® TORQUE DEVICE

POIGNEE DIRECTIONELLE

Usable Guidewire diameter / Diametre du guide : 0.010"-0.038"

ENGLISH

INSTRUCTIONS FOR USE

Slip the RADIFOCUS TORQUE DEVICE over the exposed end of the guide wire.

When the RADIFOCUS TORQUE DEVICE is in the desired position on the guide wire, tighten the nut to clamp the handle in place.

The RADIFOCUS TORQUE DEVICE may be repositioned or removed by loosening the nut and retightening it.

PRECAUTIONS

- Read instructions for use before use.
- Single use — Do not resterilize or reuse.
- Sterile and non-pyrogenic in unopened, undamaged package.
- Do not use the RADIFOCUS TORQUE DEVICE if its package is broken or soiled. The RADIFOCUS TORQUE DEVICE should be used immediately after opening the package and be disposed of properly after use.
- AVOID EXPOSURE TO WATER, DIRECT SUNLIGHT, HIGH TEMPERATURES, OR HIGH HUMIDITY DURING STORAGE.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

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 For single use only.
Strict usage unique.

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TERUMO CORPORATION 44-1, 2-CHOME, HATAGAYA, SHIBUYA-KU, TOKYO 151-0072, JAPAN MADE IN JAPAN

TERUMO MEDICAL CORPORATION SOMERSET NJ 08873, U.S.A.

Ⓢ, TERUMO and RADIFOCUS are registered in the U.S. Patent and Trademark office by TERUMO CORPORATION.

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