


RADIAL CASE PLANNING

Average vessel origin distance from left and right radial access sites based on 5'11" (180.34 cm) patient height.^{1,a}

^aAll measurements are approximations and may vary due to anatomical variances.

ARTERY DISTANCE FROM:	RIGHT	LEFT
1. Brachial	20 cm	20 cm
2. Subclavian	50 cm	50 cm
3. Brachiocephalic	70 cm	80 cm
4. Common Carotid	80 cm	70 cm
5. Celiac Trunk	95 cm	85 cm
6. Splenic	100 cm	90 cm
7. Left Gastric	100 cm	90 cm
8. Common Hepatic	100 cm	90 cm
9. Gastroduodenal	110 cm	100 cm
10. Hepatic	120 cm	110 cm
11. Renal	105 cm	95 cm
12. Superior Mesenteric	97 cm	87 cm
13. Inferior Mesenteric	110 cm	100 cm
14. Common Iliac	125 cm	115 cm
15. Internal Iliac	130 cm	120 cm
16. External Iliac	135 cm	125 cm
17. Uterine	140 cm	130 cm
18. Common Femoral Artery (CFA)	140 cm	130 cm
19. Superficial Femoral Artery (SFA)	145-165 cm ^b	135-155 cm ^b
20. Popliteal	170 cm	160 cm
21. Trifurcation	175-180 cm ^b	165-170 cm ^b
22. Anterior Tibial	185-205 cm ^b	175-195 cm ^b
23. Posterior Tibial	185-205 cm ^b	175-195 cm ^b
24. Peroneal	185-205 cm ^b	175-195 cm ^b

^bRanges reflect proximal to distal measurements of the target vessel.

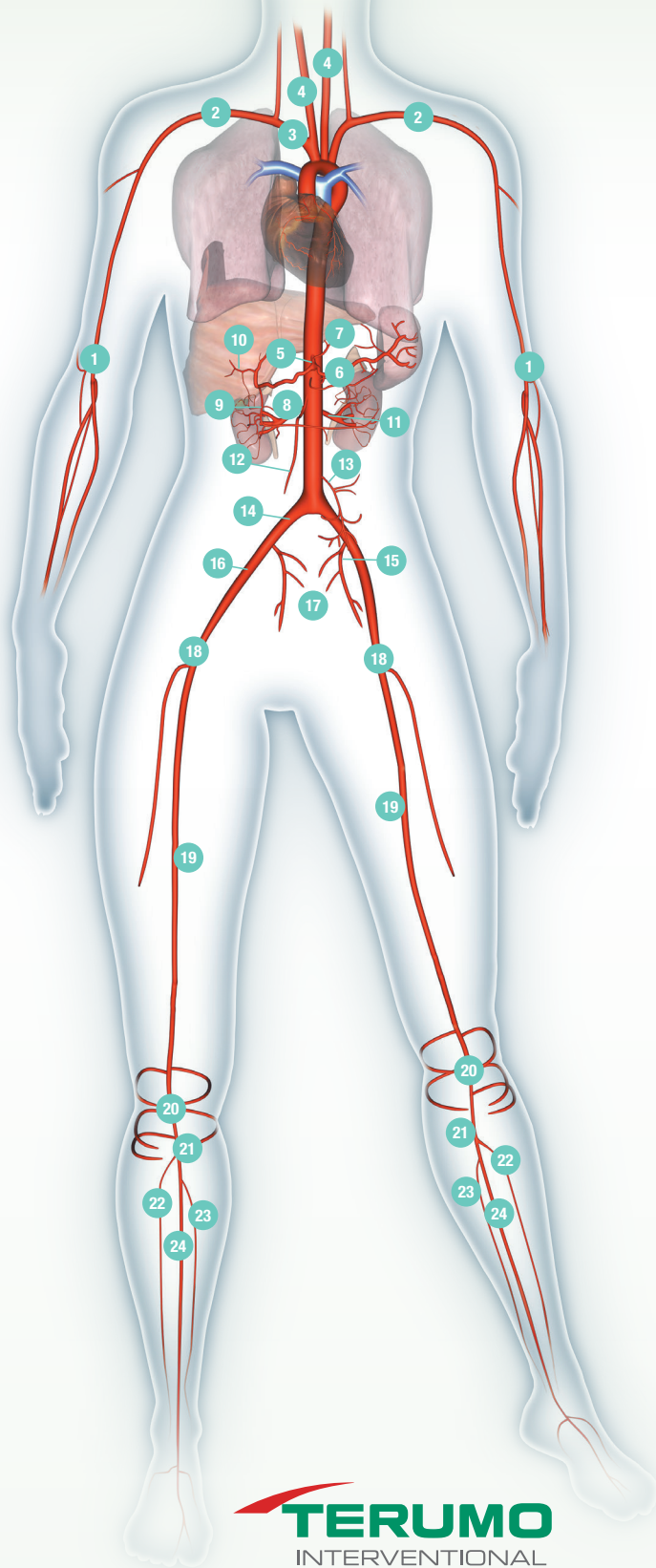

FIND OUT MORE  Phone: 800-888-3786  terumo.com

Reference:

1. Mustapha JA. When Do I Spare the Common Femoral and Access the Radial Artery? Presented at: Vascular Inter/Ventional Advances (VIVA) Annual Conference; October 2013; Las Vegas, NV.

©2025 Terumo Medical Corporation. All rights reserved. The third-party trademarks used herein are trademarks of their respective owners. All other brand names are trademarks or registered trademarks of Terumo. PM-09328